

**EZ Complement Reference Serum and Controls  
For *In Vitro* Diagnostic Use**

<b>EZ Complement Reference Serum</b>	<b>Cat. No. 789-006</b>
<b>EZ Complement Low Control</b>	<b>Cat. No. 789-008</b>
<b>EZ Complement High Control</b>	<b>Cat. No. 789-009</b>

**INSTRUCTIONS FOR THE  
RECONSTITUTION OF  
LYOPHILIZED EZ COMPLEMENT  
REFERENCE SERUM  
AND CONTROLS**

**Procedure**

1. Remove the vial(s) from the freezer.
2. Remove the cap and stopper and gently tap down all lyophilized material to the bottom of the vial.
3. Immediately add 0.3ml (300 $\mu$ l) of cold (2-8 $^{\circ}$ C) distilled water directly to the lyophilized material.
4. Replace the stopper securely.
5. Making sure that all the lyophilized material is covered with fluid, allow the vial to stand at room temperature for 5 minutes with occasional **gentle** shaking.
6. When all the material has been reconstituted at the bottom of the vial, turn the vial on its side and allow the reconstituted material to “roll around” the walls of the vial by rotating the vial slowly. (This is to make certain that any particles of lyophilized materials which may have “flaked off” are also reconstituted).

7. Invert the vial and allow the fluid to cover the stopper for 20-30 seconds, with gentle shaking.
8. Return the vial to an upright position and remove the stopper to allow any fluid which may have been trapped in the stopper to drain down.
9. Replace the stopper, shake the vial **gently**, and place it in an ice bath.
10. Allow the vial to remain on ice for 10 minutes more with occasional, **gentle** shaking.
11. Use the material as directed.

**NOTE: Once the Reference and Controls have been reconstituted they can only be used for that day's (8 hours) testing.**

**DO NOT REFREEZE THESE MATERIALS.**

**European Communities Hazardous Substance Risk Phrases  
(Regulation (EC) No 1272/2008)**

H300 –Fatal if swallowed.

H310 – Fatal if contact with skin.

EUH032 – Contact with acids liberates very toxic gas.

H410 – Very toxic to aquatic life with long lasting effect.

P264 – Wash all exposed external body areas thoroughly after handling.

P302+P352 – IF ON SKIN: Wash with plenty of water and soap.

P301+P310/P330 – IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician. Rinse mouth.

P270 – Do not eat, drink or smoke when using this product.

P501 – Dispose of contents/container as hazardous waste.

P391 – Collect spillage.

P273 – Avoid release to the environment. Refer to special instructions/ Safety Data Sheet.

Diamedix Corporation • A Subsidiary of ERBA Diagnostics, Inc.  
14100 NW 57th Court – Miami Lakes, Florida 33014 - USA  
(305) 324-2300 / (800) 327-4565  
[www.erbadiagnostics.com](http://www.erbadiagnostics.com)



Delta Biologicals S.r.l., Via Nicaragua 12/14, 00071 - Pomezia, Rome  
Italy Telephone #: +39-06-91190.1 Fax #: +39-069105244



I-789-006-9  
Rev. 4 – June 15